


*Lasts up to
1 year*

 **Juvéderm™**
INJECTABLE GEL

Instantly smooth your wrinkles

parentheses have a place
but not on your face

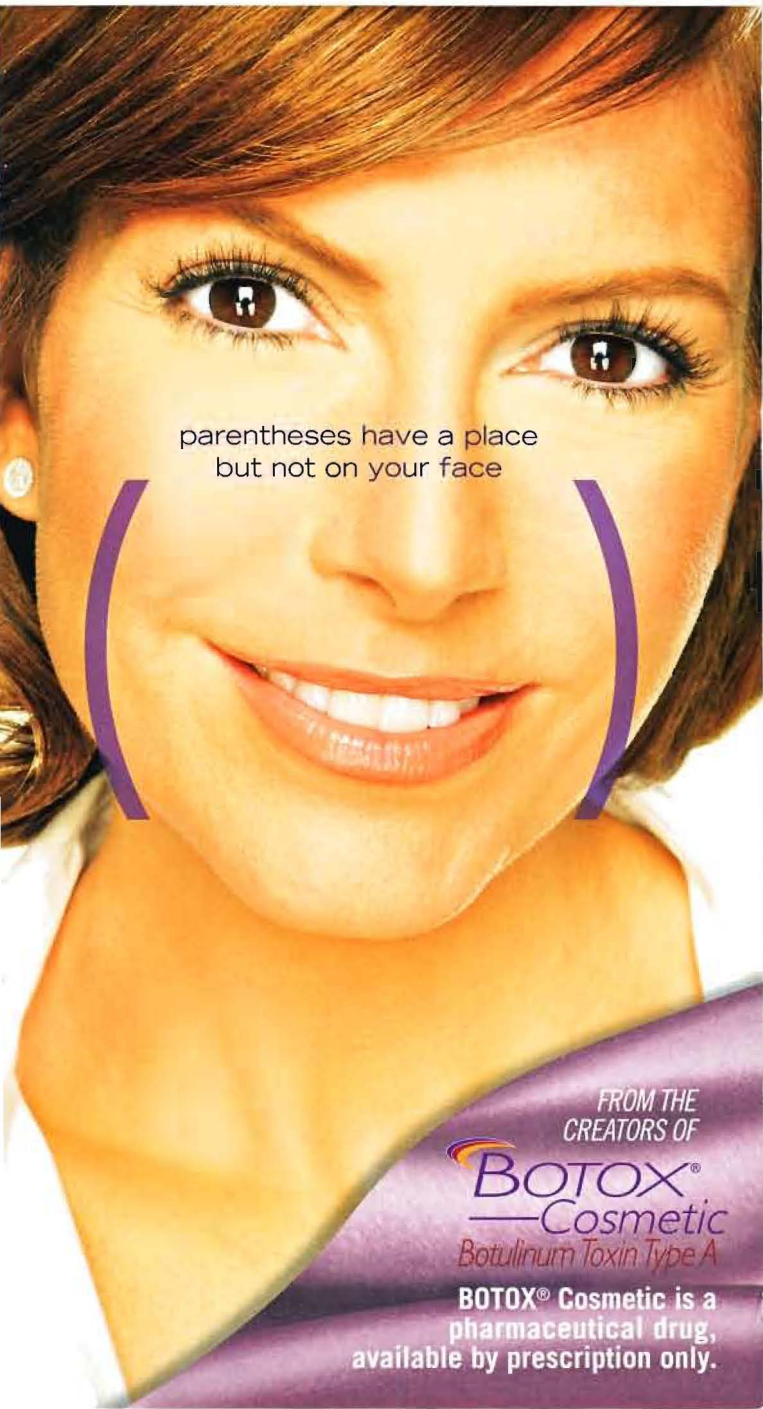

FROM THE
CREATORS OF

 **BOTOX®**
—Cosmetic
Botulinum Toxin Type A

BOTOX® Cosmetic is a
pharmaceutical drug,
available by prescription only.



How parentheses appear...

As you age, your skin changes. For example, it reduces production of a naturally hydrating substance called *hyaluronic acid* (HA). Over time, the loss of HA in your skin results in facial wrinkles and folds, like those "parentheses" lines around your mouth.

...and disappear

JUVÉDERM™ injectable gel replaces the HA your skin has lost, bringing back its volume and smoothing away facial wrinkles and folds. A breakthrough in HA dermal fillers, JUVÉDERM™ is the **first** smooth-consistency HA gel and the **only** HA filler that's FDA approved to last up to a year!*


Everyone will notice, but no one will know

You can experience the benefits of JUVÉDERM™ injectable gel in a quick, in-office procedure. Your healthcare professional eases JUVÉDERM™ under your skin to instantly smooth out the wrinkles and folds on your face. You'll see the difference immediately!

Experience the smooth,
natural look and feel
that lasts



*JUVÉDERM™ injectable gel is indicated for correction of moderate to severe facial wrinkles and folds, such as nasolabial folds.

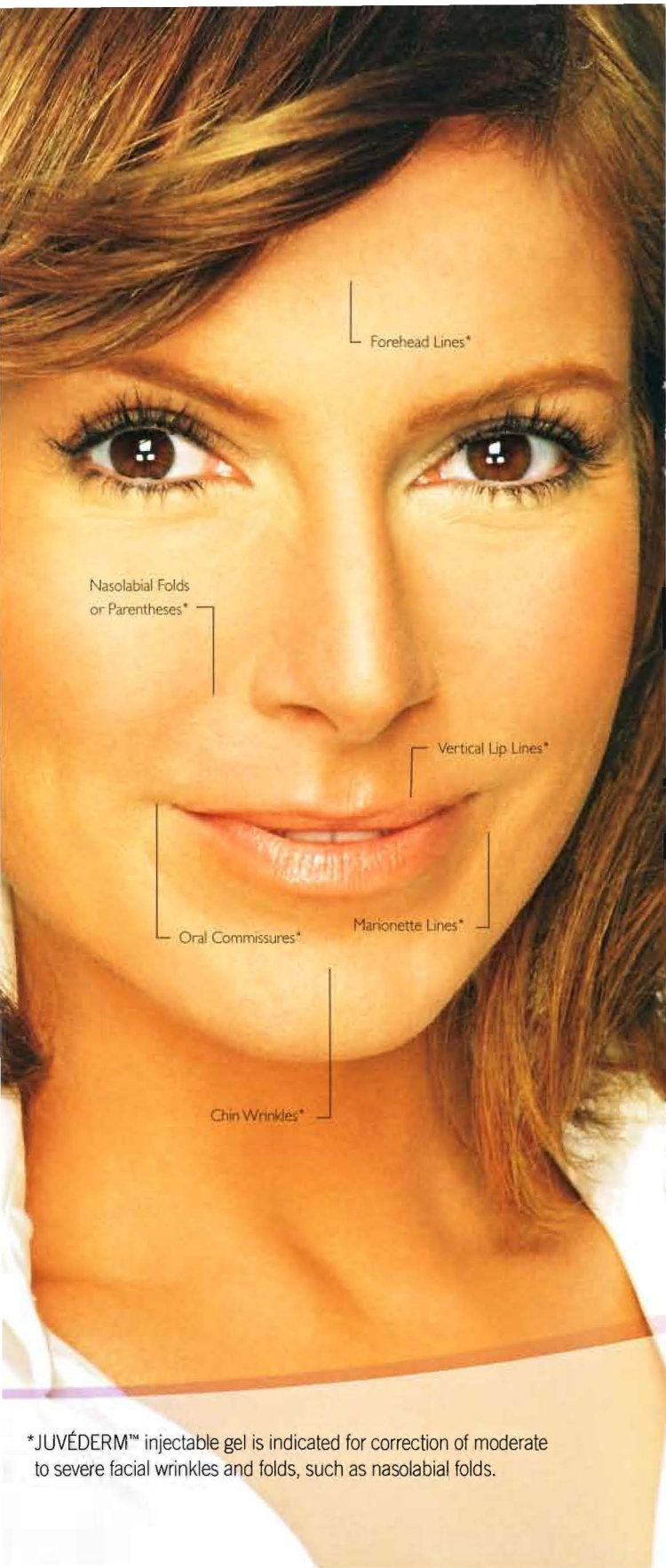

*JUVÉDERM™ injectable gel is indicated for correction of moderate to severe facial wrinkles and folds, such as nasolabial folds.

What makes JUVÉDERM™ different?

While other HA fillers on the market have a more *granular* consistency, JUVÉDERM™ injectable gel is the first *smooth*-consistency HA gel. This is due to an innovative manufacturing process called HYLACROSS™ technology. HYLACROSS™ gives JUVÉDERM™ its unique smooth-gel properties as well as an optimal gel structure that holds up over time. That's why JUVÉDERM™ injectable gel is the only HA filler that's FDA approved to last up to 1 year.*

High patient preference

When JUVÉDERM™ was tested in a clinical setting among people who didn't know which filler they were getting, nearly 90% stated a preference for JUVÉDERM™!¹ In addition, JUVÉDERM™ injectable gel has been tested on all skin phototypes, both male and female, so everyone can enjoy the benefits of JUVÉDERM™.

Are you ready to smooth out your parentheses or other facial wrinkles and folds? Ask your healthcare professional about JUVÉDERM™ injectable gel.

Everyone will notice,
but no one will know



*JUVÉDERM™ injectable gel is indicated for correction of moderate to severe facial wrinkles and folds, such as nasolabial folds.



What you can expect during and after JUVÉDERM™ injectable gel treatment

Wrinkle correction with JUVÉDERM™ is a quick, in-office procedure with immediate results and no downtime.

Common side effects with JUVÉDERM™ injectable gel usually are mild or moderate. Ask your healthcare professional how to manage them.



Before



After

1 syringe
JUVÉDERM™ Ultra 0.8 mL



Before



After

1 syringe
JUVÉDERM™ Ultra Plus 0.8 mL



Before



After

2 syringes
JUVÉDERM™ Ultra 1.6 mL



Before



After

2 syringes
JUVÉDERM™ Ultra 1.6 mL



Before



After

2 syringes
JUVÉDERM™ Ultra Plus 1.6 mL

Unretouched photographs. These photographs are not of clinical trial subjects. Individual results may vary.

Be sure to ask your healthcare professional any questions you may have about treatment with JUVÉDERM™ injectable gel.

JUVÉDERM™ injectable gel

Important Treatment Considerations for Patients

In the US, JUVÉDERM™ injectable gel is indicated for injection into the mid-to-deep dermis for correction of moderate to severe facial wrinkles and folds (such as nasolabial folds) and is generally well tolerated. The most commonly reported side effects are temporary injection-site redness, swelling, pain/tenderness, firmness, lumps/bumps, and bruising. Exposure of the treated area to excessive sun and extreme cold weather should be minimized until any initial swelling and redness have resolved. If laser treatment, chemical peel, or any other procedure based on active dermal response is considered after treatment with JUVÉDERM™ injectable gel, there is a possible risk of an inflammatory reaction at the treatment site.

Patients who are using substances that can prolong bleeding, such as aspirin or ibuprofen, as with any injection, may experience increased bruising or bleeding at injection site. You should inform your physician before treatment if you are using these types of substances. As with all skin-injection procedures, there is a risk of infection. JUVÉDERM™ injectable gel should be used with caution in patients on immunosuppressive therapy, or therapy used to decrease the body's immune response, as there may be an increased risk of infection. The safety of JUVÉDERM™ injectable gel in patients with a history of excessive scarring (eg, hypertrophic scarring and keloid formations) and pigmentation disorders has not been studied.

JUVÉDERM™ injectable gel should not be used in patients who have severe allergies marked by a history of anaphylaxis or history or presence of multiple severe allergies. JUVÉDERM™ injectable gel should not be used in patients with a history of allergies to Gram-positive bacterial proteins. The safety of JUVÉDERM™ injectable gel for use during pregnancy, in breastfeeding females, or in patients under 18 years has not been established. The safety and effectiveness of JUVÉDERM™ injectable gel for the treatment of areas other than facial wrinkles and folds (such as lips) have not been established in controlled clinical studies.

www.juvederm.com

1-877-345-5372

BOTOX® Cosmetic (Botulinum Toxin Type A)

BOTOX® Cosmetic is approved for the temporary treatment of moderate to severe frown lines between the brows in people ages 18 to 65.

Important Safety Information: Serious heart problems and serious allergic reactions have been reported rarely. If you think you're having an allergic reaction or other unusual symptoms, such as difficulty swallowing, speaking, or breathing, call your doctor immediately. The most common side effects following injection include temporary eyelid droop and nausea. Localized pain, infection, inflammation, tenderness, swelling, redness, and/or bleeding/bruising may be associated with the injection. Patients with certain neuromuscular disorders such as ALS, myasthenia gravis, or Lambert-Eaton syndrome may be at increased risk of serious side effects.

Full prescribing information has been provided to your doctor.

www.BotoxCosmetic.com

1-800-BOTOXMD